

# INSTALL GUIDE

## NEW CareLink™ Personal uploader

Upload to more easily  
partner with your doctor

### IMPROVE YOUR DOCTOR VISITS BY UPLOADING TO CARELINK™



Understand your glucose pattern




Upload regularly



Collaborate with your doctor

### GET STARTED BY DOWNLOADING THE NEW UPLOADER INSTALL FILE


A one-time download per computer where you wish to upload your device.

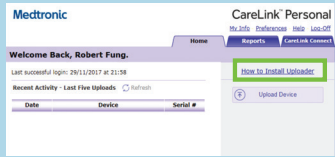

- 1 Log in to CareLink™ Personal
  - Click "How to install Uploader"




- 2 Click the download link and select a location to save the uploader file

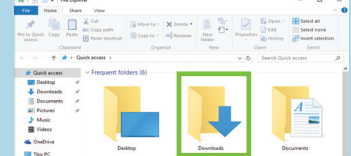


- 3 Find the downloaded file. Open the file and select "Run"


### AFTER DOWNLOADING, INSTALL THE UPLOADER

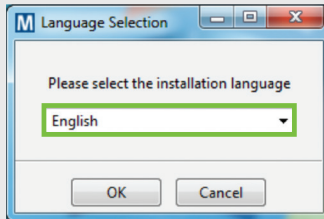


- 1 Choose your language

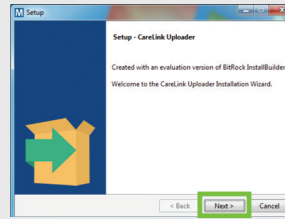

- 2 Click "Next"

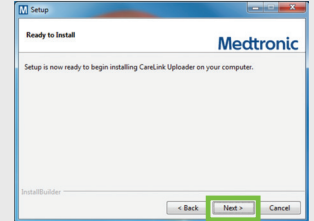

- 3 Click "Next"

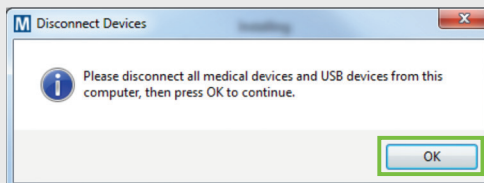

- 4
  - Make sure your CareLink™ USB or meter are not plugged in and click "OK"
  - Click "Finish" to complete the installation



### NOW, YOU'RE READY TO UPLOAD

Return to your CareLink™ Personal Homepage as you are ready to upload your Pump or Meter to CareLink™.


Please turn over for upload instructions.

Medtronic


# UPLOAD GUIDE

**NEW CareLink™ Personal uploader**  
Your first step to understanding more about your diabetes.

## INITIAL FOUR-STEP PROCESS TO ADD A NEW PUMP OR METER

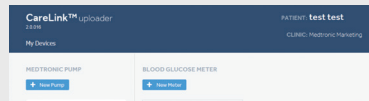

**1** Choose pump or meter

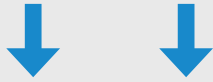


### Pump


### Meter




**2** Select the pump model

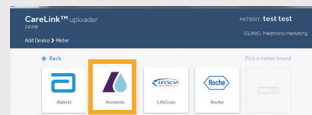


**2** Select the meter brand if you want to upload BG meter data only

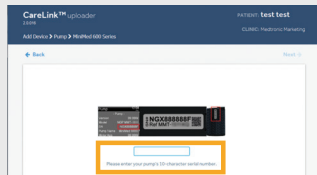


**3** Enter pump serial number

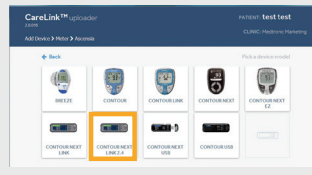

**3** Select meter model

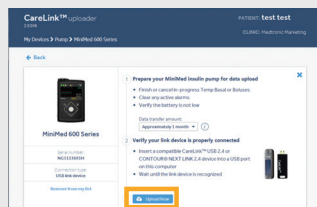

**4**  Make sure the pump is ready  
 Click "Add Pump"

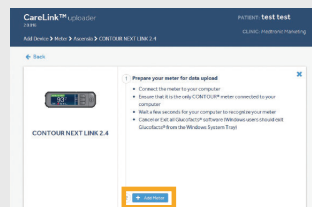

**4**  Make sure the meter is ready  
 Click "Add Meter"

## TO ENSURE SUCCESS, GET EVERYTHING READY

Plug your USB or meter into your computer

- Place your pump as close as possible to the CareLink™ USB or meter that is plugged into your computer
- Keep the meter linked with the pump



## Make sure your pump is ready to upload


Check that the pump is not on one of these screens.



- No temp basals
- No alarms
- No active bolus
- No low battery


## TWO EASY STEPS TO UPLOAD PUMP OR METER DATA

### STEP 1

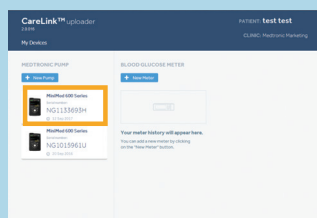

Select the device

### STEP 1

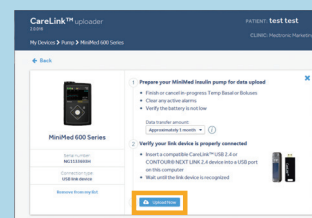

- Make sure the device is ready to upload
- Click "Upload Now"

## Safety Information: CareLink™ software

CareLink software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself.

**EUROPE**  
Medtronic International Trading Sàrl  
Route du Molliau 31  
Case postale  
CH-1131 Tolochenaz  
Switzerland  
Tel: +41 (0) 21 802 70 00  
Fax: +41 (0) 21 802 79 00  
www.medtronic.eu

## LINK WITH YOUR DOCTOR



Link to your doctor's CareLink™ system for more efficient data sharing