

# Medical Emergency Card



I am wearing a required medical device prescribed by my physician.

## I HAVE DIABETES.






If my behavior is peculiar, if I appear intoxicated, or if I am unconscious, it may be a result of severe low blood sugar.

## I AM NOT INTOXICATED. CALL A MEDICAL TEAM FOR ASSISTANCE.

If I am awake and able to swallow, give me a source of sugar (for example: juice, candy, or non-diet soft drink). Do not try to give me food or drink if I am unconscious.

## HEALTHCARE PROFESSIONALS

I am wearing an external insulin pump.

- The pump is delivering fast-acting insulin at a constant rate.
- To stop insulin delivery on the pump:
  - Press any button to wake the pump.
  - Press the Select button 
  - Unlock the pump.
  - Press the Down button  to highlight Suspend Delivery and press Select 
  - Press the Right button  to highlight Yes.
  - Press the Select button  to suspend insulin delivery.
- Do not remove the pump battery.
- Do not remove the pump without medical consent.
- If the pump is alarming, follow the instructions on the pump screen or call the local Medtronic

**In an emergency,  
please contact:**

Physician's Name:

Physician's phone number:

Family member:

Family member phone number: