



# MEDTRONIC CARELINK™ PERSONAL GUIDE BOOK

Medtronic CareLink™ Personal software

Easy to Use, Easy to Share

**PERSONALISED DIABETES INSIGHTS  
TO EMPOWER YOUR THERAPY DECISIONS**


**Medtronic**

# REGISTERING YOUR ACCOUNT


## SIMPLE PROCESS TO REGISTER YOUR ACCOUNT




- 1 Go to [www.Carelink.minimed.eu](http://www.Carelink.minimed.eu) and click on 'Create an account'. Select your country and preferred language.

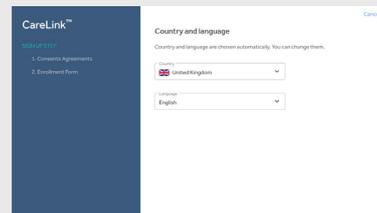


- 2 Confirm your language and country.

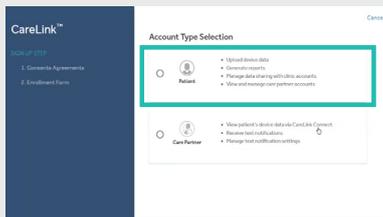


- 3 Select 'Patient' as your account type and click 'Next'.

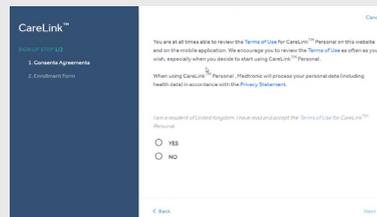

- 4 You will be taken through the consent process and your consent is required for account set up. Please ensure you scroll down to the bottom of the pages and make your selection.

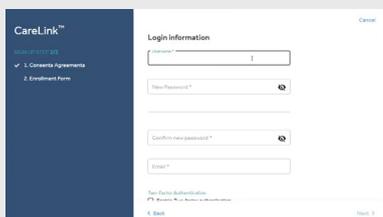

- 5 Please fill in the relevant information in the fields provided. Tip: the password must be at least 8 characters.

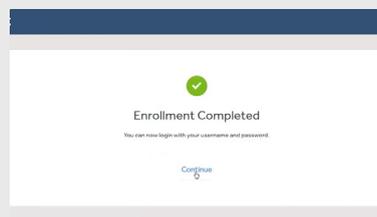

- 6 Your enrollment is now complete.



# INSTALLING THE CARELINK™ PERSONAL UPLOADER


IMPROVE YOUR DOCTOR VISITS BY UPLOADING TO CARELINK™ SOFTWARE



Upload regularly.




Understand your glucose pattern.

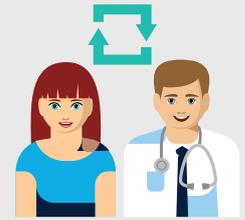

Collaborate with your doctor.

## GET STARTED BY DOWNLOADING THE NEW UPLOADER INSTALL FILE


A one-time download per computer where you wish to upload your device.

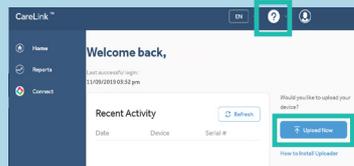

- Log in to CareLink™  
Click "How to install Uploader" OR  
Click on the "?" and then "How to install Uploader".




- Click the download link and select a location to save the uploader file.

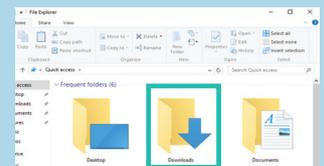


- Find the downloaded file. Open the file and select "Run".


## AFTER DOWNLOADING, INSTALL THE UPLOADER

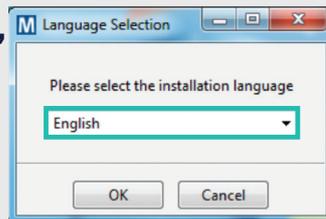


- Choose your language.

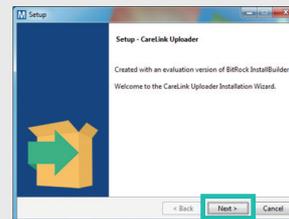

- Click "Next".

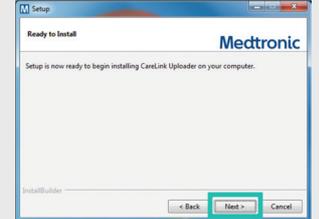

- Click "Next".

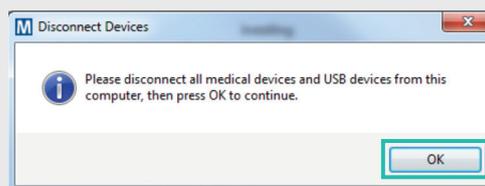

- Make sure your CareLink™ USB or meter are not plugged in and click "OK"  
Click "Finish" to complete the installation.


## NOW, YOU'RE READY TO UPLOAD

Return to your CareLink™ Personal software homepage when you are ready to upload data from your pump or meter.

## TO ENSURE SUCCESS, GET EVERYTHING READY

Plug your CareLink™ USB or meter into your computer

- Place your pump as close as possible to the CareLink™ USB or meter that is plugged into your computer
- Keep the meter linked with the pump

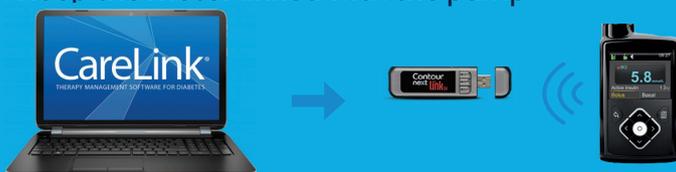

Make sure your pump is ready to upload

Check that the pump is not on one of these screens



- No active bolus
- No temp basals
- No low battery
- No alarms

# UPLOADING YOUR PUMP OR METER



**1** Click "Upload now" and wait for uploader screen to show.




**2** Click "Add new device" if this is your first upload.



**3** Select your device type if this is your first upload.

## UPLOAD METER

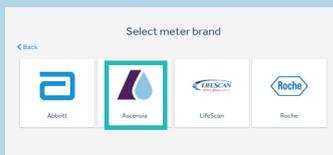

**A.** Select the meter brand if you want to upload BG meter data only.



**B.** Select meter model.



**C.** Click on 'Next' to prepare device for upload.




**D.** Wait for upload to complete.



**E.** Upload success.


## UPLOAD PUMP

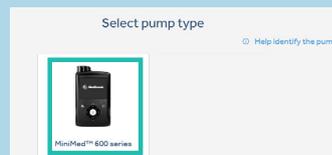

**A.** Select your pump type.



**B.** Connect link device.

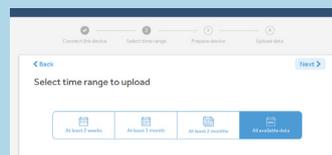

**C.** Select time range to upload.




**D.** Follow on screen instructions to prepare device for data upload.



**E.** Enter your pump serial number (first upload only).



**F.** Wait for upload to complete.

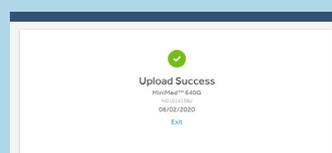

**G.** Upload success.



**4** If this is not your first upload then select the device you wish to upload on the 'Select your device type' screen.

# GENERATING YOUR REPORTS

Generating reports to manage your diabetes is made simple and accessible with CareLink™ Personal software.

- 1** Time selection.  
How many days to look back.
- 2** Set custom date range.
- 3** Switch between 30 days and 90 days view.
- 4** Click to generate reports.  
Popup will be shown.

To generate your reports click on the circles in the top left of the boxes to select the reports you wish to see.

When you know which are your favorites, click the star next to the report name and then you can click "Select Favorites" in the top left to automatically select all starred reports.

**MEDTRONIC AFRICA (PTY) LTD**  
**Waterfall Distribution Campus**  
**Cnr Bridal Veil Road & K101 Pretoria Main Road**  
**Midrand, South Africa**  
**1685**

**Tel: +27 (0) 11 260 9300**  
**E-mail: [zadiabetes@medtronic.com](mailto:zadiabetes@medtronic.com)**  
**24-hour Product Support Helpline: 0800 633786**

**[medtronicdiabetes.co.za](http://medtronicdiabetes.co.za)**

<sup>1</sup> Shalitin S, et al. Acta Diabetol. 2014; 51(2):247-56

<sup>2</sup> Corriveau EA, et al. Pediatric Diabetes 2008; 9(Part II): 360–366

Safety Information: CareLink™ software

CareLink™ software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself. For detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events, please consult the device manual.

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further. Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. UC202009935 EZ